




# ELECTRODE PLACEMENT CHART

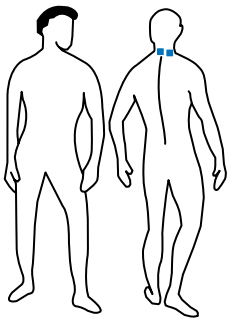


HEAD AND NECK PAIN

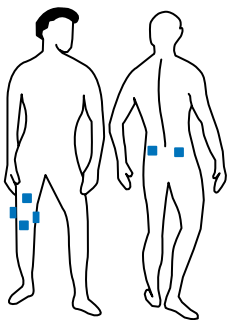


KNEE PAIN

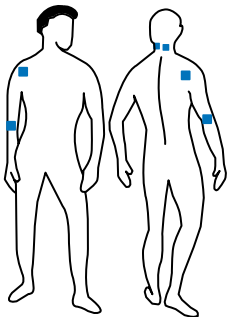


SHOULDER AND/OR ARM PAIN

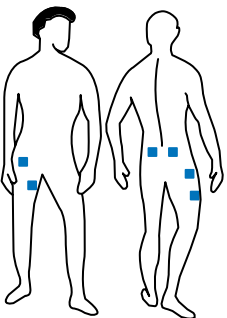


BACK WITH GROIN OR HIP PAIN

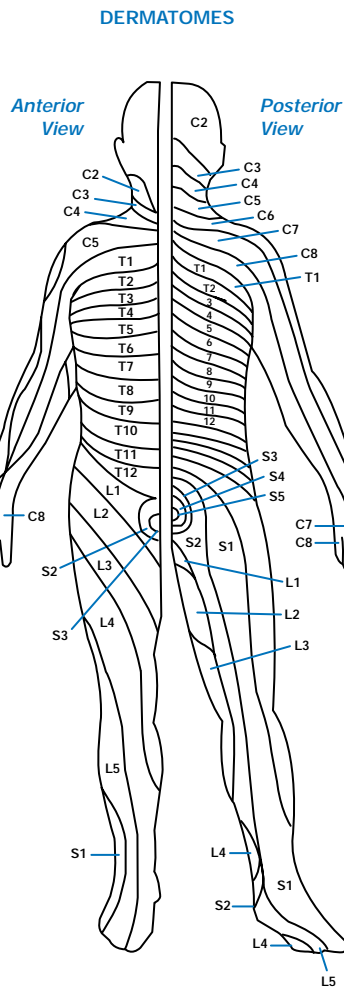


DERMATOMES

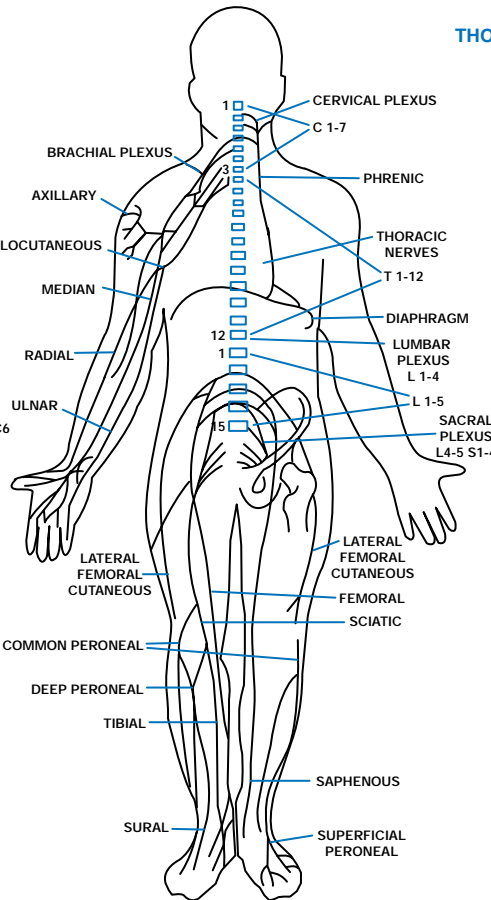


PERIPHERAL NERVOUS SYSTEM

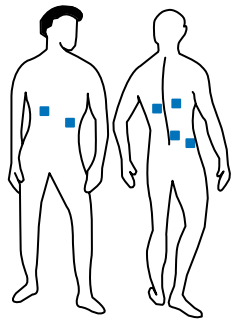


THORACIC OR INTERCOSTAL PAIN  
ALTERNATE  
USE BOTH CHANNELS

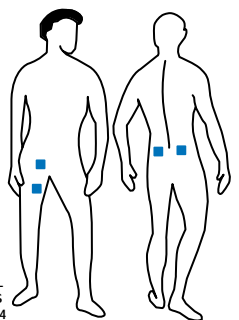


FEMORAL PAIN

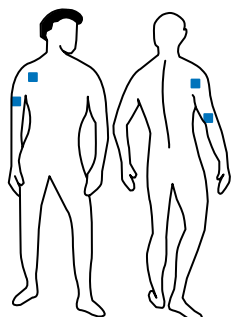


PHANTOM PAIN-  
UPPER EXTREMITIES

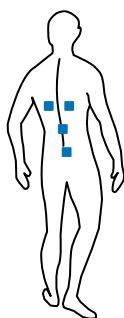


LOW BACK SACRAL OR COCCYGEAL PAIN

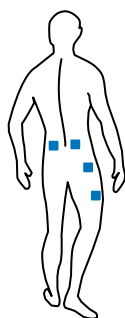


LOW BACK AND SCIATIC PAIN

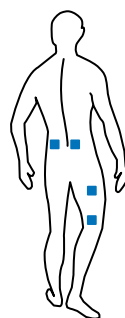

UNILATERAL LOW BACK PAIN INTO SCIATIC NERVE DOWN LEG  
ALTERNATE

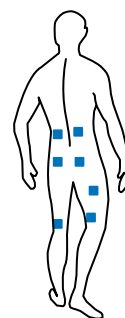


BILATERAL LOW BACK PAIN DOWN BOTH LOWER EXTREMITIES  
ALTERNATE

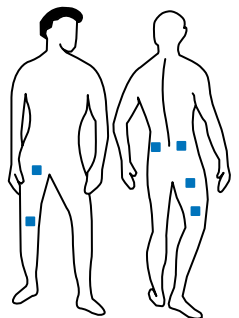

PHANTOM PAIN-  
LOWER EXTREMITIES

## NMES Electrode Placement Chart

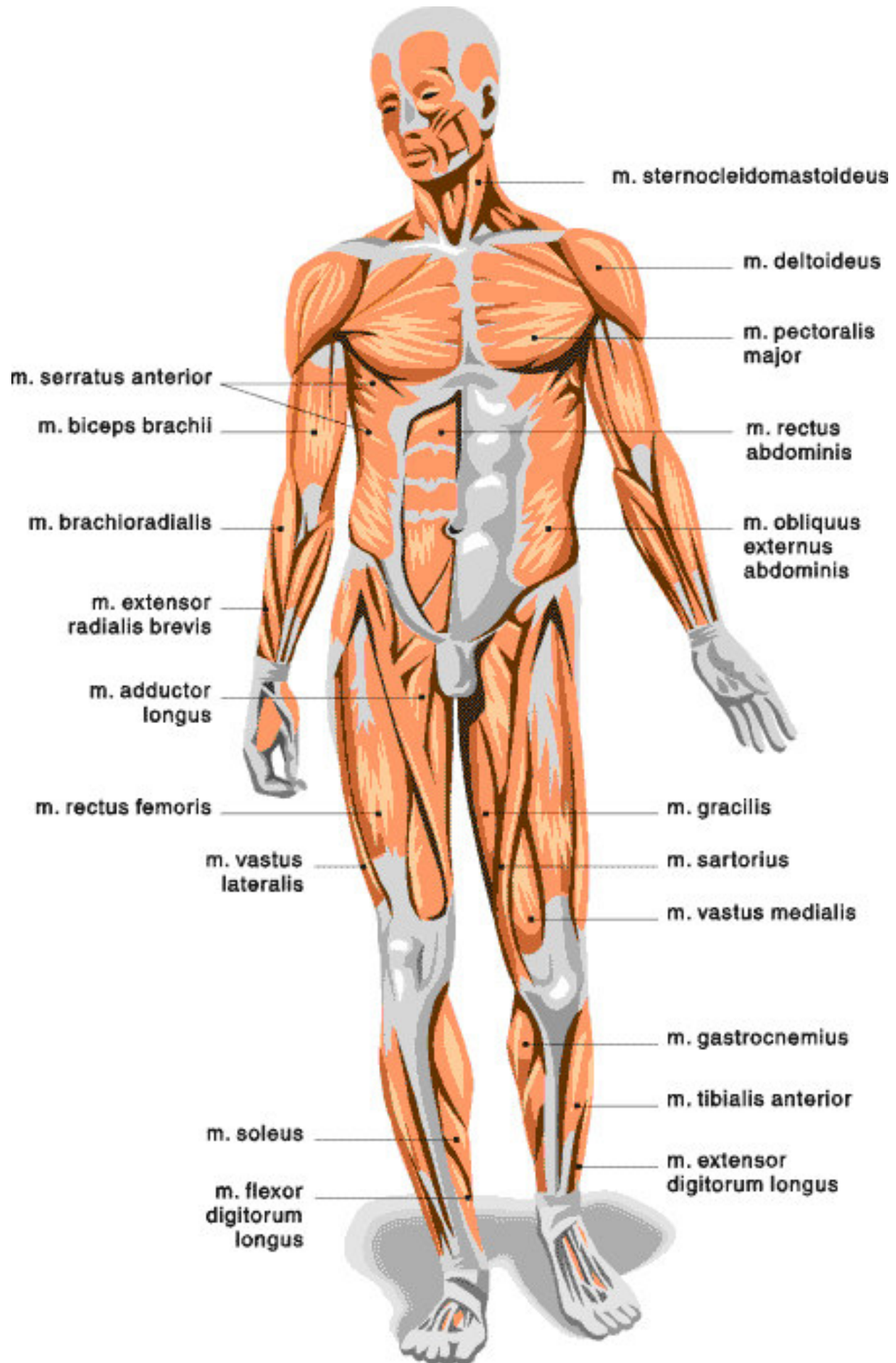

*Criterion has compiled all applicable information regarding Criterion Products for your convenience. The clinician must use his or her independent judgment when deciding which procedures and parameters are best for the patient.*

*Criterion makes no representation, guarantee or warranty, expressed or implied, that the information contained within this document is free of errors or that the use of this information will prevent differences of opinion or disputes with third-party payers, and will bear no responsibility or liability for the results or consequences of its use.*